TRANSDUCER COVER

PEELSAFE ADVANTAGE™

REFERENCE GUIDE

CE
0120

INDICATIONS FOR USE / INTENDED USE

Protective cover or sheath placed over diagnostic ultrasound transducer / probe / scanhead instruments. The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity, and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids, and particulate material to the patient and healthcare worker during reuse of the transducer (both sterile and non-sterile covers). The cover also provides a means for maintenance of a sterile field (sterile covers only). CIVCO Poly Ultrasound Transducer Covers are furnished sterile & non-sterile; single use patient / procedure, disposable.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's user guide.
- See your system's user guide for reprocessing transducer between use.
- Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.
- Dispose of single-use components as infectious waste.
- Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.
- Always place a cover over transducer to protect patients and user from cross-contamination.
- Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.

NOTE: Cover and bands are not made with natural rubber latex.

APPLYING THE COVER


NOTE: Instruction is for a two-person cover application.

1. **Sterile user:** Insert hand completely into cover. CIVCO liner should be facing upward.


2. **Sterile user:** Peel liner away from cover to expose adhesive.

NOTE: If part of cover sticks to adhesive, carefully peel it away from adhesive and proceed to next step.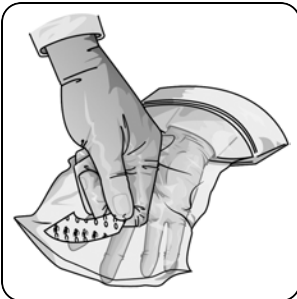

- Non-sterile user:** Apply transducer face directly to adhesive pad.
Sterile user: Grip transducer from below using hand inside cover. Use other hand to deploy cover over transducer and cord. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover.

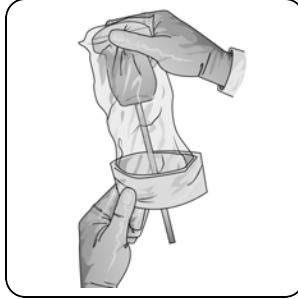**WARNING**

- Ensure no air bubbles are present prior to use. Air left between cover and transducer lens may cause poor image quality.
- Secure cover with enclosed bands.
 - Inspect cover to ensure there are no holes or tears.

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COPYRIGHT © 2017 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. PEELSAFE ADVANTAGE IS A TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH