


EXPERIENCE GUIDANCE LIKE NEVER BEFORE

 eTRAX™

*Sterile disposable
needle*

eTRAX™ Needle Tip Tracking System

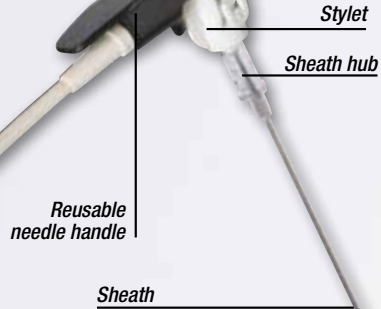

A breakthrough needle guidance system, expanding real-time image-guided navigation into a new dimension.

eTRAX features an electromagnetic sensor embedded into the needle tip. With the ability to now fuse other imaging modalities with ultrasound, eTRAX takes image fusion to a new dimension, helping give physicians a more dynamic and accurate tool for tracking the tip of a needle under real-time, image-guided navigation. eTRAX can provide significant clinical advantages over other technologies including increased clinical confidence in targeting difficult-to-access lesions, improved workflow, reduced procedure time and increased patient safety.

Now clinicians can have a highly accurate, 3D anatomical roadmap to access difficult targets. eTRAX may be used with devices for applications such as:

- Ablations
- Core tissue biopsies
- Drainage
- Fluid aspirations
- Therapeutic delivery
- Vascular access
- Anesthesia



How does eTRAX work?



1. Cover reusable eTRAX needle sensor with sterile CIV-Flex™ sensor cover



2. Slide sterile disposable needle sheath over eTRAX needle assembly and snap into place



3. After puncture, remove sensor and disposable stylet from needle sheath

16GA – For use with SONIXTOUCH

Accepts instruments 18GA or smaller

REF. #	PRODUCT DESCRIPTION
610-1055	eTRAX Starter Kit with non-sterile reusable needle sensor (1), sterile disposable eTRAX needle, 16GA x 17.7cm (7") and 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover (5) (Ref.#410-251-16)



REF. #	PRODUCT DESCRIPTION	QTY.
610-1606-10 (610-1606-01)	Sterile eTRAX needle, 16GA x 6cm (2.4") with 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10
610-1608-10 (610-1608-01)	Sterile eTRAX needle, 16GA x 8cm (3.1") with 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10
610-1610-10 (610-1610-01)	Sterile eTRAX needle, 16GA x 10cm (3.9") with 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10
610-1613-10 (610-1613-01)	Sterile eTRAX needle, 16GA x 13cm (5.1") with 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10
610-1615-10 (610-1615-01)	Sterile eTRAX needle, 16GA x 15cm (6") with 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10
610-1057 (610-1056)	Sterile eTRAX needle, 16GA x 17.7cm (7") and 7.6 tapered to 3.8 x 147cm (3" tapered to 1.5" x 58") CIV-Flex needle cover	10

*To order eTRAX Starter Kit contact your local Analogic representative

*To reorder eTRAX replacement kits contact CIVCO: 800-445-6741





Accurate Guidance. Confident Outcomes.

For international distribution, visit CIVCO Worldwide at www.CIVCO.com

Worldwide Headquarters:

102 First Street South, Kalona, Iowa 52247-9589 USA

319/656-4447, 800/445-6741 · Fax: 319/656-4451, 877/329-2482

www.CIVCO.com

COPYRIGHT © 2014. CIVCO IS A TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ETRAX IS A TRADEMARK OF CIVCO. ALL OTHER TRADEMARKS ARE THE PROPERTY OF THEIR RESPECTIVE OWNERS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2014R-2808