

# TEE TRANSPORT TRAY

Simplify Your Workflow and Stay Compliant with the TEE Transport Tray



Simplify your TEE probe cleaning process by contacting **800.445.6741** to order your **TEE Transport Tray**

REF #	PRODUCT DESCRIPTION	QTY
610-1418	TEE Probe Transport Tray	30

## Compliant Workflow

Reversible lid for easy **clean/dirty** indication

## Protect Your Investment

Keep your expensive, delicate probe safe from damage by protecting the distal tip

## Prevent Cross-Contamination

Safeguard disinfected probe shaft from the handle, cord and connector

## OEM Approved

Meet probe manufacturer transducer coiling requirements

## Simplified Process

Eliminate time and resources needed to clean reusable tray

## Stay Clean, Be Green

Made from recycled materials



[CIVCO.com/TEE-Transport](http://CIVCO.com/TEE-Transport) | [info@civco.com](mailto:info@civco.com) | 800.445.6741

COPYRIGHT © 2018. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2018R-3058 REV 1

# TEE TRANSPORT TRAY WORKFLOW

Use the same tray to transport to and from the exam room with ease



- 1** Stack clean trays near probe storage. Transport clean probe in clean tray to exam room.
- 2** Post-exam, place dirty probe in same tray & flip lid to dirty. Transport probe to reprocessing location and dispose of tray.
- 3** Use a clean tray to transport the disinfected probe to its storage cabinet.

Visit [CIVCO.com/TEE-Transport](https://www.civco.com/TEE-Transport) to view the TEE Transport Tray workflow video



CIVCO.com | info@civco.com | 800.445.6741

